



SHIPPING INSTRUCTIONS

FOR QUESTIONS OR ASSISTANCE,
CALL THE VERISTRAT SUPPORT HOTLINE AT 866-432-5930

✓ If sample **cannot be immediately** shipped, store at room temperature.
Sample must be shipped within **24 hours** of collection.

1




Place **serum collection card** into foil bag and seal.

4



Insert **completed FedEx airbill** into pouch on VeriStrat shipping envelope.

2

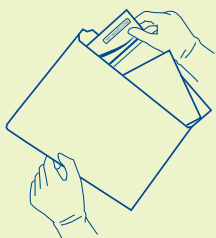

Place foil bag and **physician signed Test Request Form** into VeriStrat shipping envelope.

5



Call FedEx at
1-800-GOFEDEX
(1-800-463-3339) for pickup.

3



Complete **FedEx airbill**.
If shipping on Friday,
check Saturday delivery.
Keep top copy.

Biodesix assumes that each laboratory has adopted processes and procedures for handling blood, including those related to blood-borne pathogens, phlebotomy and personal protective equipment, including without limitation, standards developed by the Clinical and Laboratory Standards Institute and, in particular CLSI H4-A5, *Procedures and Devices for the Collection of Diagnostic Capillary Blood Specimens*, 2004. Please follow your institution's approved guidelines and the respective material safety data sheets (MSDS). © 2010 Biodesix. All rights reserved. 302010 08/10